**Fluid/Gas Tubing Set**  
**REF TR 9006-11**

**DIRECTIONS FOR USE**

*(English)*

***Circulating Nurse:***

1. Peel open outer pouch, letting inner pouch drop onto sterile field. Use caution not to touch inner pouch.

***Scrub Nurse:***

2. Peel open sterile pouch, dropping tray with tubing onto sterile field.
3. Remove tubing coil from tray.
4. Hand off filter end of the tubing set to circulating nurse.

***Circulating Nurse:***

5. Connect filter end of tubing set to appropriate fluid/gas transfer system.

***Scrub Nurse:***

6. Pretest functionality by placing luer end of tubing set into water or saline solution and observing air flow.
7. After assurance of functionality, connect luer end of tubing set to female end of infusion cannula.

**WARNING**

1. This sterile disposable medical device should not be reused. After use, all parts of this disposable pack, as well as any remaining fluid, should be properly disposed of. 
2. If any component in this pack is received in a defective condition, do not use and notify Escalon immediately.

***Please direct inquiries to:***

Escalon Medical Corporation  
2440 South 179th Street  
New Berlin, WI 53146 USA  
Tel 800-433-8197  
262-821-9182  
Fax 262-821-9927

***Authorized European representative:***

Drew Scientific Limited Co.  
Sowerby Woods Industrial Estate  
Barrow in Furness  
Cumbria LA14 4QR  
United Kingdom  
Tel: +44 1229 432089

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