



## Irrigating Contact Lens

**REF** TR9840-11 Flat Lens

**REF** TR9841-11 Minus Lens

### INDICATION FOR USE

Assists in visualization of surgical field while providing means for irrigation.

### DIRECTIONS FOR USE

***Circulating Nurse:***

1. Peel open the outer pouch letting the inner pouch drop on to the sterile field. Use caution not to touch the inner pouch.

***Scrub Nurse:***

1. Peel open the sterile pouch, place irrigating contact lens onto the sterile field by turning tray up-side-down.
2. Break and remove paper tape from lens steel tube.
3. Connect female adaptor fitting of tubing to male adaptor end of irrigation solution.
4. Run irrigation solution through contact lens until all air bubbles are clear of the lens.
5. The irrigating contact lens is now ready for use.

### WARNINGS

1. This sterile disposable medical device should not be reused. After use, this product should be treated as potentially biohazardous and should be properly disposed of.
2. If any portion of this product is received in a defective condition (including packaging), do not use and notify Escalon immediately.



**STERILE**

**Escalon Medical Corporation**

2440 South 179<sup>th</sup> Street • New Berlin, WI 53146 • USA

Tel 800-433-8197 • 262-821-9182 • Fax 262-821-9927

[www.escalonmed.com](http://www.escalonmed.com)

5539 Rev. E (ECO021071)