



**TORNAMBE EYE LEVEL  
TR9049-11  
DIRECTIONS FOR USE**

- 1.) Remove the protective cover from the adhesive back.
- 2.) Place the eye level over the patients eye patch with the air bubble at the 12 o'clock hour.
- 3.) Position patient so that the air bubble is at the clock hour suggested by the surgeon.

Escalon Medical Corp  
2440 South 179<sup>th</sup> Street  
New Berlin, WI 53146  
(800) 433-8197  
(262)-821-9182  
Fax (262) 821-9927